Inhalt
Content
Table des Matieres
Contenuto
Indice
Inhoud

D	Bedienungsanleitung Bitte vor Gebrauch sorgfältig lesen!	1
GB	Instruction Manual Please read carefully before use!	23
F	Mode d'emploi A lire attentivement avant d'utiliser l'appareil!	45
I	Istruzioni sull'uso Legere per favore attentamente prima dell'uso!	67
E	Instrucciones de uso Por favor, léalas detenidamente antes del primer uso!	89
NL	Gebruiksaanwijzing Voor gebruik zorgvuldig lezen s.v.p.!	111

HEARTBEAT

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HEARTBEAT

**A watch that measures
pulse rate and blood pressure
- Health is fun!**

Instructions for use

Home Diagnostic Products GmbH International

D-52222 Stolberg

Visit our Internet web site at: www.hdp.de

Copyright © Revision 1, April 1999 HDP GmbH, all rights reserved.
Duplication or publication of the contents without written permission
from HDP GmbH is prohibited.

Contents

1	Functional components	26
2	Introduction	27
2.1	Preliminary remarks	27
2.2	Items supplied	27
2.3	CE mark	27
2.4	Manufacturer's declaration	27
3	Instructions for use	28
3.1	Information on blood pressure	28
3.2	Intended use	28
4	Using the "Heart Beat"	29
4.1	Preparation	29
4.2	Putting on the "Heart Beat"	29
4.3	Measuring the pulse rate and blood pressure	30
4.4	Saving the reading	30
4.5	Preventing the reading from being saved	30
4.6	Setting and activating the timer	31
4.7	Calling up stored data (display)	32
4.8	Setting time and date	33
	Description of keys on "Heart Beat"	34
4.9	Transferring stored data to PC	35
4.10	Deleting stored data	35

Contents

5.	Software	36
5.1	Computer-assisted evaluation	36
5.2	Installation	36
5.3	System requirements	36
5.4	User restrictions	36
6	Explanation of symbols	37
7	Important notes	38
7.1	Precautionary measures	38
7.2	Important note	38
8	Care and maintenance	39
8.1	Care	39
8.2	Changing the cuff	39
8.3	Calibration check	39
9	Error classification and correction	40
9.1	Error numbers	40
9.2	Description of error	41
10	Warranty and repair conditions	42
10.1	Warranty	42
10.2	Repair	42
11	Technical data	43

I Functional components

- ① START/STOP key
- ② MENU key
- ③ Infrared eye
- ④ Cuff
- ⑤ Battery compartment at top of unit on back
- ⑥ Display

2 Introduction

2.1 Preliminary remarks

This wristwatch for pulse and blood pressure measurement provides a top-quality system incorporating the latest technology, which is easy to use and extremely reliable. Thank you for your confidence in us; please do not hesitate to contact us at any time if you have queries concerning the product or servicing.

2.2 Items supplied

1. "Heart Beat" wristwatch pulse and blood pressure measurement device for home use
2. Instructions for use
3. Two batteries
4. Heart Beat software (CD-ROM)

2.3 CE mark

"Heart Beat" conforms to the European specifications on which medical products legislation is based. The quality of the equipment has been examined under EC regulation 93/42/EEC and bears the CE mark CE 0434.

2.4 Manufacturer's declaration

Manufacturer:

I.E.M. GmbH
D-52222 Stolberg

The "Heart Beat" brand name complies with the relevant requirements of EC regulation 93/42/EEC concerning medical products issued by the Council on 14 June 1993.

Stolberg, 1 April 1999

3 Instructions for use

3.1 Information on blood pressure

The circulatory system in our bodies supplies all organs and tissues with sufficient oxygen and nutrients to meet their requirements. The heart acts as a pump to keep the blood moving. It normally beats to a regular rhythm, 60 to 80 times a minute. This is known as the heart rate or pulse rate.

Blood pressure is subject to a number of changes in the course of the day and during the night. When sleeping, blood pressure is low. High to very high values occur when we are agitated or in periods of physical exertion. The temporary increase in blood pressure at these times is quite normal. It soon goes down again.

Blood pressure is the result of contraction and relaxation of the beating heart. The upper (systolic) and lower (diastolic) pressures can be measured. Blood pressure readings are expressed in millimetres of mercury (mmHg).

**The upper limit of the normal range for adults
is 135 / 85 mmHg.**

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If we measure blood pressure regularly, we can learn a great deal about the way our own bodies work and their reactions to outside influences.

We can therefore lead healthier lives!

Why not find out your own blood pressure patterns? Your doctor will certainly be pleased to help.

3.2 Intended use


- The "Heart Beat" is to be used in measuring pulse rate and blood pressure.
- To obtain accurate values, the watch should be worn on the wrist under any clothing. When taking a reading, it is important to hold it level with the heart.
- The watch is suitable for wrists with a circumference of between 14 and 19.5 cm.
- The watch is suitable for measuring pulse and blood pressure in adults only.
- Never make any change to your current medication on the basis of the results you obtain. The assessment of results and decisions regarding subsequent treatment can only be taken by the doctor.**

4 Using the "Heart Beat"

4.1 Preparation


Open the battery compartment on the back of the watch. Insert the batteries supplied as shown in the diagram. Make sure that the battery removal strip is located under the batteries.

Then set the time and date (see 4.8). The "Heart Beat" is now on permanent standby.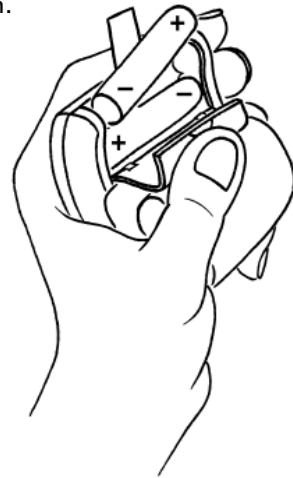

4.2 Putting on the "Heart Beat"

Put the "Heart Beat" on like a normal watch so that the time is visible. Fasten the strap securely around your wrist. Remember to hold the watch **level with your heart** so that you can read the display. Readings will be incorrect if it is held too high or too low. To obtain accurate readings and avoid errors, the arm, hand and fingers must be **kept still** when taking a measurement.

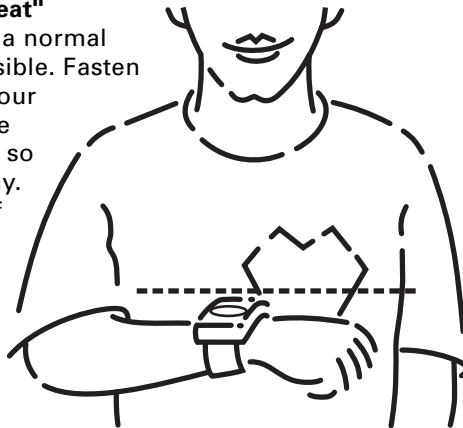4 Using the "Heart Beat"

4.3 Measuring the pulse rate and blood pressure

Press the START/STOP key once. A short display test will appear:

Measurement then begins. The "Heart Beat" gradually inflates and deflates. The measurement takes approx. 30 seconds to complete. The result is displayed:

After the measurement has been completed, the display automatically returns to the time and date.

The measurement process can be interrupted at any time by pressing the START/STOP key.

4.4 Saving the reading

The reading is automatically saved to memory. Do not touch any key while the measurement number is flashing (approx. 10 seconds). Up to 50 measurements can be saved. After this, each subsequent reading replaces the "oldest" in the memory.

4.5 Preventing the reading from being saved

The measurement number (21 in the diagram) is flashing. Saving of the reading can be prevented by pressing the START/STOP key within 10 seconds.

Important

We recommend that blood pressure readings should always be taken while sitting down, after a rest of approx. 3-5 minutes.

4 Using the "Heart Beat"

4.6 Setting and activating the timer

The timer function allows pulse and blood pressure values to be taken at specified intervals between 1 minute and 59 minutes – useful when determining your own daily pattern or when exercising.

Press the MENU key once until the bell symbol begins to flash:

Pressing the START/STOP key leaves the minutes display (00) flashing. Each additional time the START/STOP key is pressed the number of minutes increases by 1. Holding the key down increases the numbers automatically. The timer is activated after 30 seconds or immediately after pressing the MENU key and begins the countdown.

When zero is reached, an acoustic signal is heard for 20 seconds, which can be stopped by pressing any key. The "Heart Beat" display returns automatically to the time and date. A measurement can now be started by pressing the START/STOP key.

The timer function can be interrupted by pressing the MENU key briefly.

4 Using the "Heart Beat"

4.7 Calling up stored data (display)

Up to 50 measurements are stored automatically by the "Heart Beat".

Press the MENU key twice (approx. 1 second). The last reading saved and your measurement number (21 in the diagram) will appear on the display. Pressing the START/STOP key will make the display alternate every 2 seconds between the values measured and the time of the measurement:

Pressing the START/STOP key again will scan through the stored data. If no key is pressed during the next 30 seconds, the memory display will stop automatically or it can be stopped immediately by pressing the MENU key.


**This function is available only after readings have been saved to memory!
If no data has been stored, the display will go directly to the "setting time and date" function.**

4 Using the "Heart Beat"

4.8 Set time and date

Press the MENU key 3 times (approx. 1 second) until the symbols for month, day, hour and minute start to flash.

If no readings have been stored to memory, this function appears after pressing the MENU key twice. The minutes, hours, year, month and day displays can now be set.

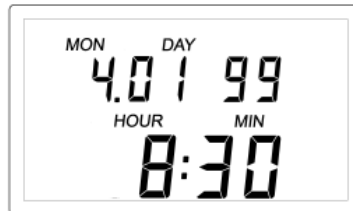Press the START/STOP key once and the minutes will flash. Pressing again will advance the flashing digits in the minutes field by 1.

If the START/STOP key is held down, the number increases automatically.

Now press the MENU key to call up the various digits to display the hours and the date. Use the START/STOP key to set the correct numbers.

If the MENU key is pressed once more after setting the day, or if no entry is made for 30 seconds, the time and date are updated.

4 Using the "Heart Beat"

4 Using the "Heart Beat"

4.9 Transferring stored data to PC

If the "Heart Beat" software is installed on your PC you can evaluate the stored "Heart Beat" readings. For simple data transfer in seconds, the "Heart Beat" has an infrared eye.

Hold the START/STOP key down. If the MENU key is also pressed within 5 seconds, the display will show "co". The device is now in communication mode.

Place the "Heart Beat" approx. 10 cm from the infrared eye of your computer. If your computer does not yet have an infrared eye for data capture, this is available from HDP GmbH as an accessory. Please contact your dealer if necessary. Data transfer can only be triggered via the software.

End the communication mode ("co" display) by pressing any key once.

4.10 Deleting stored data

If the START/STOP key is held down for more than 10 seconds, the stored measurements are deleted. After 5 seconds the "Clr" display appears and an acoustic signal alerts you to the imminent deletion.

5 Software

5.1 Computer-assisted data evaluation

The "Heart Beat" software can be used to evaluate the readings of up to 10 individuals simply and clearly and to keep long-term records.

Records can be displayed or printed out in a variety of ways, e.g. in tabulated or chart form. The program also offers interesting information associated with blood pressure. The entire "Heart Beat" manual is again included on the CD-ROM, ready to print out. The software is available in Windows 95 or 98 versions.

Your readings can be entered manually or using the infrared eye of the "Heart Beat" for transfer of the stored data in seconds.

For larger groups of users, the "Heart Beat Professional software" is more suitable. This is available as an extra from HDP GmbH.

5.2 Installation

1. Place the CD-ROM in the drive
2. Start SETUP.EXE on the CD-ROM
3. Follow the instructions in the set-up program

Once installation is complete, a new program directory will have been set up.

5.3 System requirements

Pentium PC or above, Windows 95 or 98, 16 MB RAM (32 MB recommended) SVGA card (65536 colours), CD-ROM drive (at least quadruple), mouse, sound card recommended.

5.4 User restrictions

You are entitled to use the software for private purposes only. If it is to be used for commercial purposes, a special agreement with HDP GmbH is necessary. This applies both to the "Heart Beat software" and also to the "Heart Beat Professional software". By installing the software on your computer you agree to abide by the contractual conditions shown on the CD-ROM with regard to use of the software and manual.

6 Explanation of symbols

Symbol	Operation/situation	Meaning/solution
	Pulse located	
SYS [mmHg]	Systolic reading display	top value
DIA [mmHg]	Diastolic reading display	bottom value
PULSE [1/min]	Pulse display	Number of beats per minute
	Timer function	appears when timer function is activated
HOUR	Hours display	
MIN	Minutes display	
DAY	Day display	
MON	Month display	
	Battery low	Change battery (see 4.1)
Err 01	Defect report	see 9.1
No	Measurement no.	
Clr	Delete measurement data	If the START/STOP key is further 5 seconds, all measurement is deleted
co	Communication mode	Data transfer via infrared eye possible

7 Important notes

7.1 Precautionary measures

If discomfort is experienced during a measurement, press the START/STOP key to deflate the cuff immediately. Unfasten the cuff and remove the device. Please contact your dealer.

7.2 Important note

Please also note the following:

- If you suffer from cardiac arrhythmias (irregular heart beats) or are pregnant you should consult your doctor to check whether this method of measuring your own blood pressure with the "Heart Beat" is suitable for you.
- This unit is not suitable for use in cases of severe arteriosclerosis of the wrist, advanced arterial occlusive disease or in conditions involving vascular spasm.
- The pulse display on the unit cannot be used for checking the heart rate of a cardiac pacemaker.
- On no account use the Heart Beat when bathing or showering. Should moisture penetrate the unit, the batteries must be removed immediately and further use of the unit avoided. Contact your dealer.
- The unit should not be operated in the vicinity of an MRI scanner.
- The use of components not supplied with the equipment can lead to incorrect measurements (except for batteries).

8 Care and maintenance

8.1 Care

If necessary, clean the plastic housing and cuff with a slightly moistened cotton cloth.

Important note:

The cuff of the "Heart Beat" must not be washed or dry cleaned.

**Batteries should not be discarded with domestic waste!
Please dispose of batteries only in the disposal bins provided in shops.**

8.2 Changing the cuff

If you need a new cuff for your "Heart Beat", this can be obtained from HDP GmbH as an accessory. **For technical reasons, the cuff should only be changed by HDP GmbH.** Please consult your dealer if necessary.

8.3 Calibration check

The equipment is calibrated by the manufacturer for a two-year period. The calibration check must therefore be performed at least every two years.

This service is offered at an additional charge by HDP GmbH, a competent authority or by authorised maintenance services – in accordance with the legislation governing the operation of medical measuring equipment.

9 Error classification and correction

9.1 Error numbers

Number	Cause	Correction
Err 01	Movement artefact	Hold arm still and level with heart and repeat measurement
Err 02	Cuff	<ul style="list-style-type: none">• Put on cuff correctly (see 4.2)• The wrist circumference must be between 14 cm and 19.5 cm
Err 03	System error Pneumatic defect	Do not take further readings and contact dealer
Err 04	Battery failure	Change battery (see 4.1)

9 Error classification and correction

9.2 Description of error

Error description

During the measuring process, pressing the START/STOP key once does not immediately interrupt the measurement and deflate the cuff without delay.

START/STOP key pressed but the required measurement not completed or pumping process discontinued prematurely

No symbols appear on display

The battery symbol appears on the display

Time and/or date incorrect

Incorrect measurements

Error correction

Unfasten cuff, remove battery and do not use unit. Contact dealer.

Check batteries and change if necessary

Check batteries and change if necessary

Change batteries

See 4.8: Setting the time and date

- Put on cuff as specified (see 4.2)
- The wrist circumference must be between 14 cm and 19.5 cm
- Hold arm still while measuring
- Hold arm level with heart while measuring

10 Warranty and repair conditions

10.1 Warranty

1. HDP GmbH provide a one year warranty on the blood pressure measuring unit from the date of purchase. The correctly completed warranty card or invoice constitute evidence of the date of purchase.
2. Deficiencies due to material defects or manufacturing errors will be corrected free of charge within the warranty period.
3. Services provided under warranty do not extend the period of warranty, either for the unit itself or for selected components.
4. The following are excluded from the warranty:
 - a) all damage resulting from incorrect handling, e.g. failure to observe the instructions for use.
 - b) damage attributable to repair or intervention by the retailer or unauthorised third parties.
 - c) Damage occurring in transit between the manufacturer and the user or during return to the customer service department.
 - d) Accessories subject to normal wear and tear (cuff, batteries, etc.).
5. Liability for direct or indirect damage caused by the equipment is also excluded even if the damage to the unit is recognised as covered by the warranty.
6. Lost warranty cards cannot be replaced. Warranty cards altered without justification are invalid.
7. Further claims, for whatever reasons these are made, are excluded.

HDP provides no warranty for the batteries supplied with the equipment.

10.2 Repair

In the case of a warranty claim, please complete the warranty card with your exact name and address and full details of the fault. Please contact your dealer for information on methods of dispatch.

II Technical data

Measurement range	Systole (SYS): 70–260 mmHg Diastole (DIA): 45–180 mmHg Pulse (PULSE): 40–240 (bpm)
Display range	0–300 mmHg Pulse: 40 – 240 beats per minute (bpm)
Measurement method	Oscillometric method
Storage capacity	50 measurements (systolic and diastolic pressure, pulse rate, date and time)
Accuracy	Pressure: ± 3 mmHg in display range
Data security	Non-volatile memory
Data transfer	Infrared
Power source	2 alkaline 1.5V micro-batteries (AAA, LR03)
Measuring time	approx. 30 seconds
Controls	Two multifunction keys: START/STOP and MENU
Display	Liquid crystal display (LCD) visible size approx. 22 x 37 mm
Operating temperature	+10 °C – +40 °C
Operating humidity	15% – 90%
Storage temperature	-20 °C – +50 °C
Storage humidity	15% – 95%
Cuff size	Wrist strap Wrist circumference between 14 and 19.5 cm
Dimensions	74 x 55 x 27 mm (W x D x H)
Weight	90 g without batteries
Read-out software	for Windows 95, 98

Right to technical modifications reserved

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Copyright © Revision 1. April 1999 HDP GmbH, alle Rechte vorbehalten.
Copyright © Révision 1er avril 1999 HDP GmbH, tous droits réservés.
Copyright © Revisione 1° aprile 1999 HDP GmbH, tutti i diritti riservati.
Copyright © Revisie 1 april 1999. HDP GmbH. Alle rechten voorbehouden.
HDP GmbH geeft geen garantie op de meegeleverde batterijen.